USER MANUAL  
BRUKSANVISNING

# IBIS XP

---

---

**Engels ..... 4**

**Norsk..... 27**

---

# English

© 2008 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Handicare.

The information provided is based on general data concerning the constructions known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Handicare cannot be held liable for possible errors in the information or the consequences thereof. Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

2008-01

---

<b>Introduction</b> .....	<b>6</b>
This manual .....	6
Product reference .....	6
Service and technical support.....	6
Identification plate .....	6
Approval .....	7
CE declaration .....	7
Warranty .....	7
Liability .....	9
Used wheelchairs and the environment.....	9
<b>1 Safety</b> .....	<b>10</b>
1.1 General safety instructions.....	10
1.2 Decals and instructions on the wheelchair .....	10
1.3 Electronic safety.....	10
<b>2 General description</b> .....	<b>11</b>
2.1 Product description Ibis XP .....	11
2.2 Main components.....	12
2.3 Controller .....	13
<b>3 Operating instructions</b> .....	<b>14</b>
3.1 Inspection before use.....	14
3.2 Getting in and out.....	14
3.3 Free wheel lever .....	14
3.4 Switching on the wheelchair.....	15
3.5 Driving .....	15
3.6 Curves .....	16
3.7 Obstacles.....	16
3.8 Slopes .....	17
3.9 Reducing the wheelchair.....	17
3.10 Transport.....	19
3.11 Adjustment options (dealer) .....	20
3.12 Electrical adjustment options .....	22
<b>4 Maintenance</b> .....	<b>23</b>
4.1 Maintenance table.....	23
4.2 Batteries .....	23
4.3 Checking and inflating the tyres .....	24
4.4 Cleaning the wheelchair.....	24
<b>5 Troubleshooting</b> .....	<b>25</b>
5.1 Troubleshooting table.....	25
<b>6 Technical specifications</b> .....	<b>26</b>

---


## Introduction

### This manual

With this manual you can safely use and maintain (clean) the wheelchair. This manual should be used in combination with Handicare's Sedeo seating system manual (supplement). In case of doubt always contact your dealer.

**Strictly follow the procedures with a caution symbol! These procedures can - if they are not executed properly - result in human injury or damage to the product or the environment.**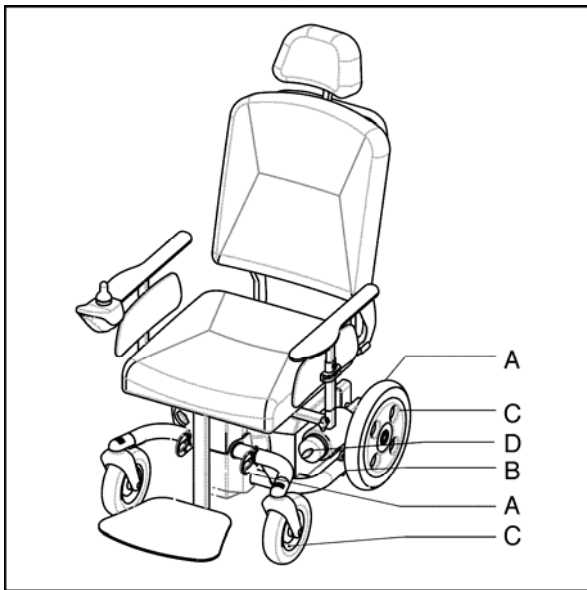


### Product reference

- A Transport brackets
- B Identification plate
- C Tyre pressure rear wheels/front wheels
- D Free wheel

### Service and technical support

For information concerning specific settings, maintenance or repair work, please contact your dealer. Make sure you always mention the model, year of manufacture and identification number. This information is provided on the identification plate of the product.

A service manual is available for the dealer.

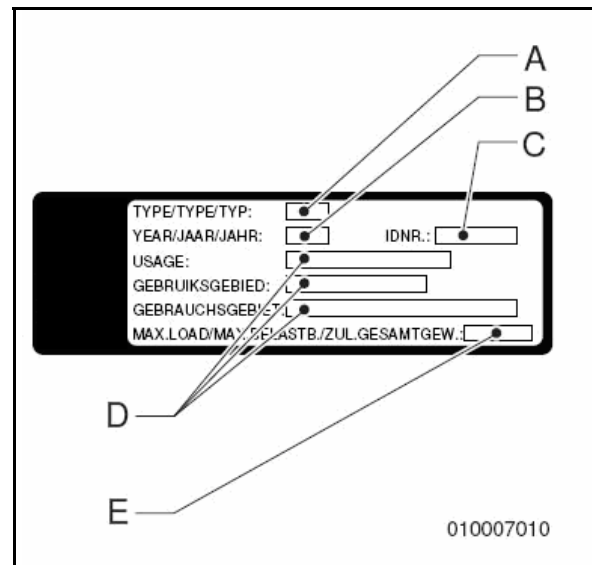### Identification plate

- A Model
- B Year of manufacture
- C Identification number
- D User area indoors or outdoors
- E Maximum load in kg

## Approval

The product meets the following standards:

1. EN12184 Electrically driven wheelchairs and scooters, class B.
2. ISO7176-8 Requirements for impact, static and fatigue strength.
3. ISO7176-9 Climate tests for electric wheelchairs and scooters.
4. ISO7176-14 Standards and test methods for controller systems of electrically driven wheelchairs.
5. ISO7176-16 Requirements of resistance to fire.
6. ISO 7176-19 Crash test.

The product is EMC (Electro Magnetic Compatibility) approved in accordance with EN12184.

## CE declaration

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

## Warranty

In the following warranty and liability stipulations the terms and definitions as summed up hereafter must be explained as follows:

- **Product:** The hand-operated or electric wheelchair or scooter manufactured and delivered by Handicare.
- **Customer:** The person who directly obtains a Product from Handicare.
- **Dealer:** The person who delivers a Product obtained from Handicare to customers or third parties.
- **User:** The person who uses a Product manufactured by Handicare.

Irrespective of what is determined concerning warranty conditions applicable to the Product, in any case the following applies with regard to the warranty:

1. Except insofar as described otherwise Handicare guarantees the Product for its suitability for the purpose for which the Product is intended - all of these points as described in this manual - and for the quality of the material of which the Product is made and the manner in which the Product is manufactured.
2. Repairs or replacement of parts of the Product that may be necessary as a result of faults that are based on qualitatively faulty material or manufacturing errors will be executed free of charge, as long as such faults occurred within one (1) year after the date of delivery of the Product to the Customer. The parts to be replaced must be shipped post-paid to Handicare. Disassembly or assembly of these parts is for the expense of the Customer. Therefore the following cases are not eligible for free repair or replacement as meant above:
  - The repair or replacement that is necessary in connection with faults that arise after one (1) year after the date of delivery of the Product to the Customer;
  - The repair or replacement that is required in connection with faults due to improper or careless use of the Product or that are based on the Product being used for another purpose than the one for which it is intended, in which regard it applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against

## Introduction

---

- possible claims from Users or other third parties for faults based on an incorrect or careless use of the Product;
- Parts that are subject to wear, and the need for repair or replacement of the parts are the actual consequence of normal wear.
3. Irrespective of that stipulated under 2, it applies as far as an electric Product is concerned that with regard to the battery that forms part of the Product warranty is only given in case of faults or non-functioning of the battery that are demonstrably the direct consequence of material or manufacturing errors. A fault or non-functioning of the battery as a result of normal wear is not covered by the warranty as meant in these warranty stipulations. Similarly not covered by the warranty are faults or non-functioning that are the consequence of improper or unprofessional use of the Product or the battery that is part of the Product, including the incorrect charging of the battery and the failure to carry out timely and good maintenance, in which context it also applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against possible claims from Users or other third parties that are based on the above mentioned improper or unprofessional use of the Product or the battery that is part of the Product.
  4. The warranty conditions as mentioned in above stipulations in any case become null and void if:
    - The guidelines of Handicare for the maintenance of the Product are not, or insufficiently, followed;
    - A necessary repair or replacement of parts is based on neglect, damage or abuse of the Product or a use of the Product for another purpose than the one for which it was intended;
    - Parts of the Product are replaced by parts of another origin than those which Handicare uses and/or parts of the Product are replaced without the permission of Handicare.
  5. The warranties as mentioned in stipulations 1 through 3 also become null and void if what is involved is re-used by a new user within the warranty period and that such re-use necessitates adaptations to the product and those adaptations are carried out without the instructions and/or at the order of Handicare.
  6. To retain rights under the above delineated warranties the Customer must, in case of damage or other calamities, as rapidly as possible contact Handicare and inform them as fully as possible. The possibility of taking recourse to the above-mentioned warranty conditions becomes null and void for the Customer in any case after 20 workdays after the claim situation or the calamity arises that was the reason for the recourse to the guarantee.
  7. The replacement of a part or the repair or the reconditioning of the Product within a current warranty period does not extend the warranty period.
  8. Handicare gives no warranty on repair to or reconditioning of the Product carried out other than under order of and/or at the instructions of Handicare. If repairs and/or reconditioning are executed

by or on behalf of a Customer, the Customer indemnifies Handicare with respect to the claims of third parties who result, in the broadest sense of the word, from such repairs or reconditioning.

electronic components that must go to electronic waste. Used batteries belong to chemical waste.

### Liability

Irrespective of what is determined regarding liability in the general conditions applicable to the Product, with regard to liability in any case the following applies:

1. Taking into consideration the following stipulations, Handicare only accepts liability for loss due to death or physical injury that is the result of a defect in the Product for which Handicare is responsible and for damage to another object that is the private property of the user of the Product, as long as said loss is the direct result of a fault in the Product.
2. Handicare accepts no other or further liability than delineated under 1. In particular Handicare accepts no liability for consequential damage, in any form whatsoever.

### Used wheelchairs and the environment

If your wheelchair has become superfluous or needs to be replaced, it can usually be taken back by your dealer. If this is not possible, please contact your local authorities for the possibilities of recycling or an environmentally friendly way of disposing of the used materials.

For the production of the wheelchairs several plastics and metals have been used. The wheelchair also contains

# 1 Safety

Handicare accepts no liability whatsoever for damage or injury caused by the failure to (strictly) adhere to the safety guidelines and instructions or else due to carelessness during the use and cleaning of the wheelchair and possible accessories. Depending on the specific work circumstances or the accessories used, additional safety instructions may be required. Please contact your dealer immediately if you observe a potential danger during the use of the product.

## 1.1 General safety instructions

- **The user of the wheelchair is at all times fully responsible for the fulfilment of the locally applicable safety regulations and guidelines.**
- **The technical specifications must not be altered.**
- **Modifications to (parts of) the wheelchair are not permitted.**

## 1.2 Decals and instructions on the wheelchair

- **Signs, symbols and instructions placed on this wheelchair comprise part of its safety facilities. They must never be covered or removed and must remain present and clearly legible throughout the entire lifespan of the wheelchair.**

- **Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your dealer with regard to this.**

## 1.3 Electronic safety

- The standard version of your electric wheelchair has been tested according to the strictest EMC requirements. Mobile telephones have no influence on the driving behaviour of the wheelchair. When using a mobile telephone in the vicinity of an electric wheelchair with special adaptations, you are advised to first switch off the electric wheelchair.
- Your electric wheelchair may influence electromagnetic fields such as alarm systems.
- If the electronics of the wheelchair is not properly screened off, this may affect sensitive electrical appliances, such as shop alarms and garage openers. The electric wheelchair has been tested for this. Should any problems of this nature occur, please report these to your dealer immediately.

## 2 General description

### 2.1 Product description Ibis XP

The Ibis XP is a compact electric wheelchair with joystick controller for use in and around the house. The Ibis XP is available as front wheel drive or rear wheel drive.

Due to its compact design, the Ibis XP is highly manoeuvrable. The frame height of the wheelchair can easily be adjusted, ensuring an ideal seat height for everyone. Additionally, the seating system can be fully customized to satisfy the specific wishes and requirements of the individual.

The Ibis X-Series has been designed for:

- Transport of persons to a maximum weight of 160 kg.
- Use on paved roads, sidewalks, foot- and bicycle paths.
- Use in and around the house.

The Ibis XP is part of the Ibis X-Series, a range that extends from a push wheelchair (Ibis XC), a manual or foot-propelling wheelchair (Ibis XA) to an electric model (Ibis XP). As a result, a wheelchair offering maximum comfort and safety is available for every situation.

Your dealer must give you good driver's instructions before you can independently operate the product and participate in traffic.

You must be able to correct the consequences of actions while driving the wheelchair.

Consider operating the wheelchair under the supervision of an experienced dealer during first operations.

The wheelchair is not a motorised vehicle in the sense of traffic regulations.



The Ibis XP has a maximum speed of 6 kph.

If you use the wheelchair for other purposes than those for which it is intended, Handicare accepts no liability whatsoever for damage or injury resulting from such use other than for which the wheelchair was developed and designed.

**Please study this manual carefully before you use the wheelchair.**

## General description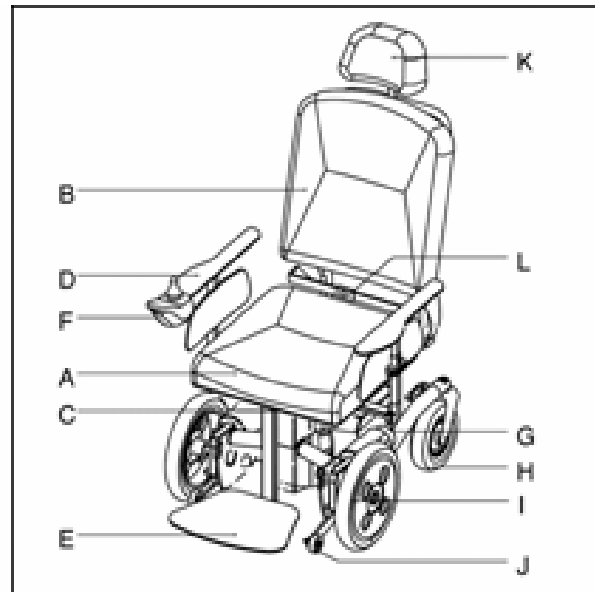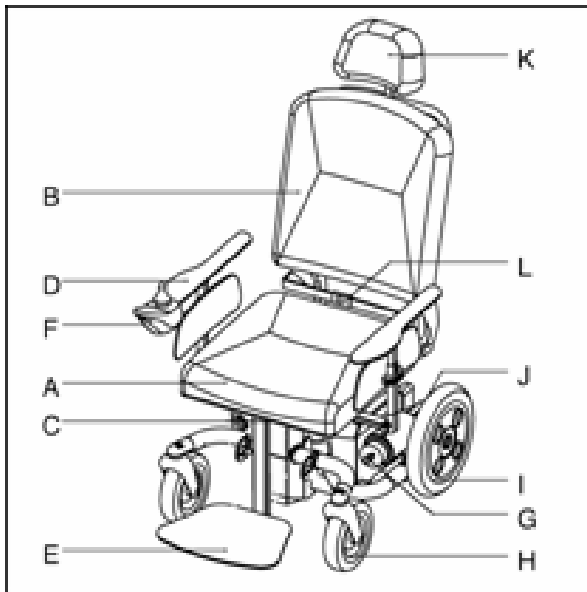

## 2.2 Main components


- A Seat
- B Backrest
- C Seat angle adjustment
- D Armrest
- E Legrest
- F Controller/charge connection for the batteries

- G Free wheel
- H Castor wheels
- I Drive wheels
- J Anti-tip brackets (option)
- K Headrest (option)
- L Lap belt, to be provided by the dealer (option)

## 2.3 Controller

The controller contains the following components:

- A On/off key
- B Joystick
- C Charge connection
- D Horn
- E Battery indicator
- F Speed control
- G Indicators
- H Lighting
- I Warning lights
- J Actuators/adjustment options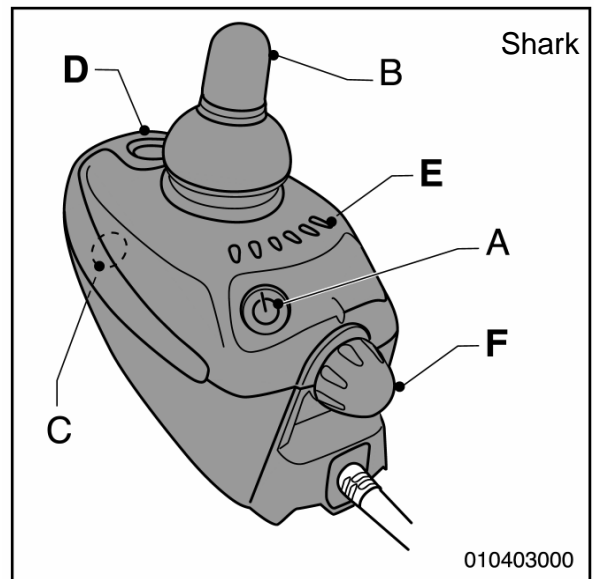

**Performance adjustments should only be made by professionals conversant with the adjustment process and the operator's capabilities. Incorrect settings could cause injury to the operator or bystanders, or damage to the vehicle or surrounding property.**

The joystick is for driving the wheelchair and adjusting the various electrical options, if installed.



### Battery indicator

The battery indicator shows the condition of the batteries.

When all the lights are on, the batteries are fully charged. As the batteries are depleted, lights from right to left will go out.

**If only the red lights are on, the batteries must be charged immediately.**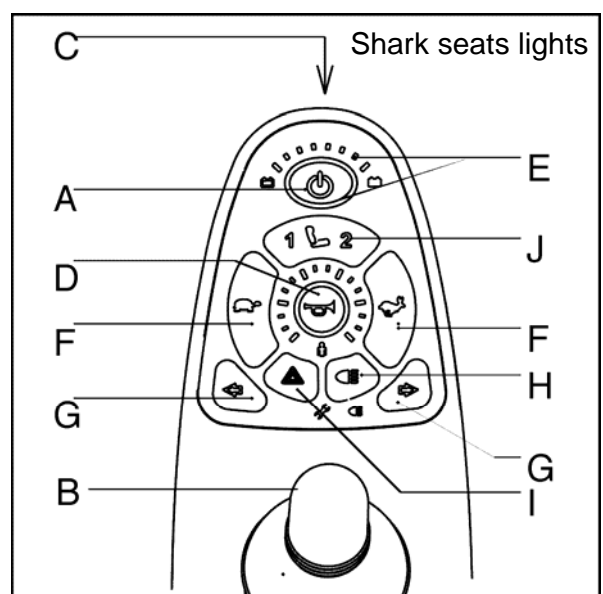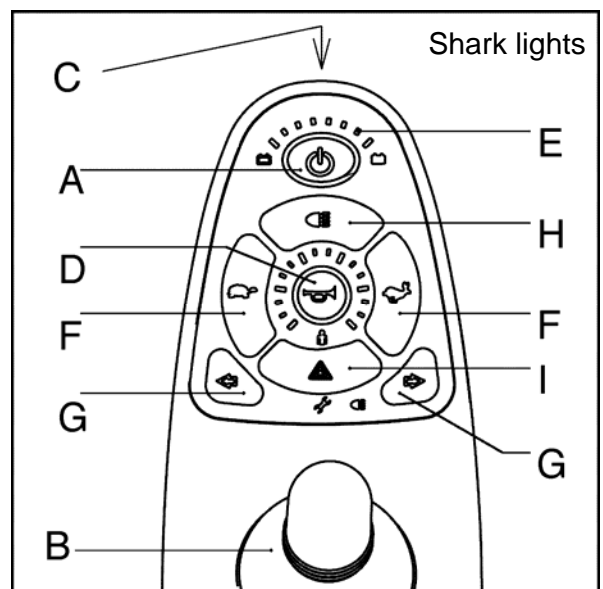

## 3 Operating instructions

### 3.1 Inspection before use

It is important to check the following points before you start driving the wheelchair:


- If the lights and indicators are working properly.
- If the tyres are sufficiently inflated.
- If the batteries are charged sufficiently. The green battery indicator lights must be on.

**In winter the batteries have a lower capacity. In case of light frost the capacity will drop to approximately 75% of the normal capacity. At a temperature of less than -5° the capacity will be approximately 50% of the normal capacity. This will affect the range.**

### 3.2 Getting in and out

1. Make sure the wheelchair is switched off electrically.
2. Put the free wheel lever in the Drive position.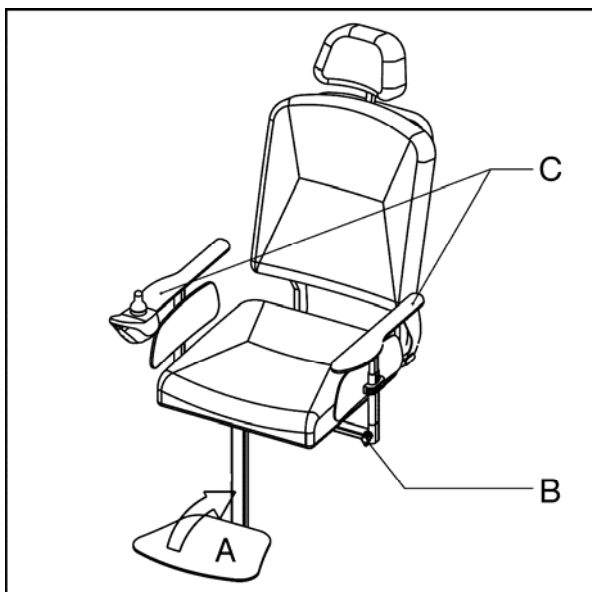

### Transfer forwards

1. Fold up the foot support (A)
2. Seat yourself in the chair.

### Transfer sideways

1. Loosen the cap (B) and remove the armrest (C).
2. Seat yourself in the chair.


**Switch off the wheelchair electrically before you get in or out! This way you will avoid that the wheelchair starts to move when you accidentally move the joystick.**

### 3.3 Free wheel lever

Put both free wheel levers in the DRIVE position (A) to be able to drive the wheelchair electrically.

To be able to push the wheelchair, put both free wheel levers in the **PUSH** position.

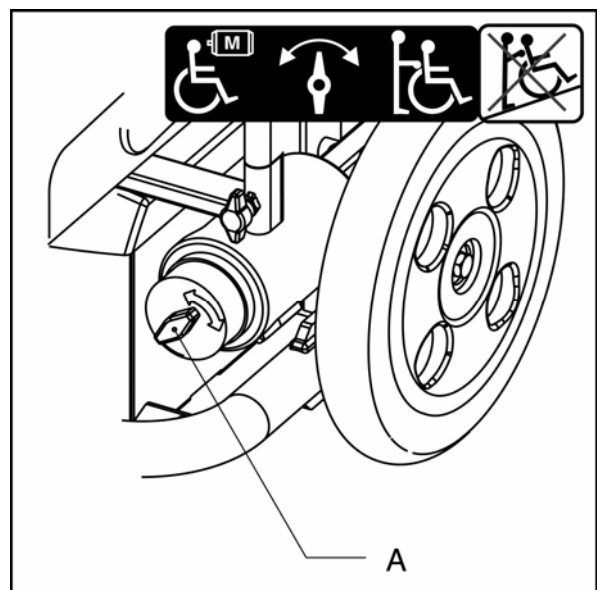
- Never put the wheelchair in the Drive position when you are parked on a slope. When the wheelchair is parked on a slope that is too steep, it can start moving down the slope.
- After pushing, the free wheel lever should immediately be returned to the Drive position.
- The free wheel lever should only be put in the Push position if the wheelchair is to be pushed. In the Drive position the automatic parking brake is activated.

### 3.4 Switching on the wheelchair

To switch on the wheelchair, push the on/off key (A).

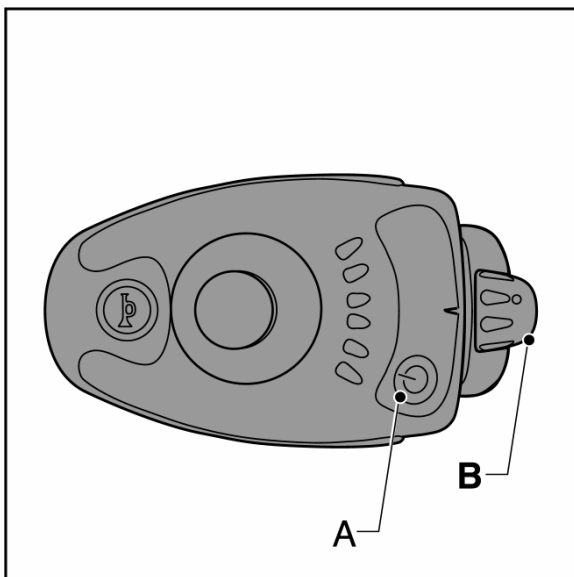When an electronics error is detected, the battery indicator will start flashing. The electronics are blocked.

### 3.5 Driving


- As a wheelchair user you could be vulnerable in traffic. When travelling in or across a road, keep in mind that other traffic participants may not always notice you. Keep to the applicable traffic regulations.
- Avoid out-of-the way routes, so that help can be on its way quickly when needed.

#### Driving (Shark)

1. Switch on the wheelchair.
2. Use the speed control (B) to select the desired speed.
3. Slowly push the joystick in the desired driving direction. The further you push the joystick, the faster you will drive.

#### Driving (Shark lights and Shark seats-lights)

1. Switch on the wheelchair.
2. Use the speed control (F) to select the desired speed.
3. Slowly push the joystick in the desired driving direction. The further you push the joystick, the faster you will drive.

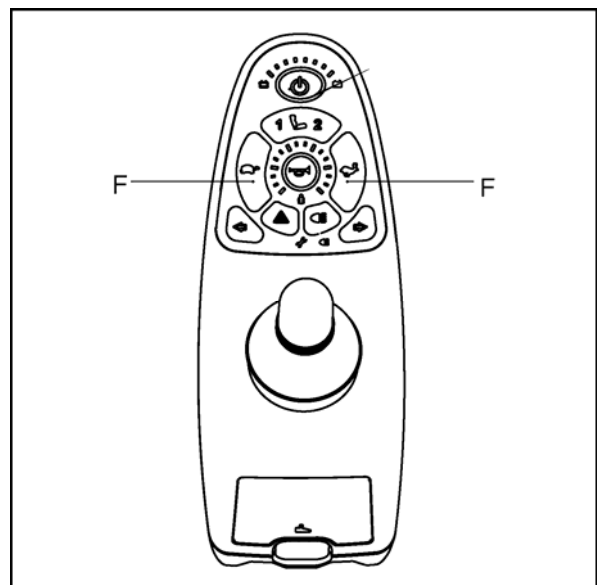### Braking

When you release the joystick or move it into the central position, you will brake automatically.

### Emergency stop

1. When driving forward pull the joystick all the way backward with one move. When reversing push the joystick all the way forward with one move.
2. Immediately release the joystick when the wheelchair has come to a standstill.

### Miscellaneous


- Drive carefully on slippery roads, resulting from rain, ice or snow!
- Prevent the wheelchair from coming into contact with seawater: seawater is caustic and may damage the wheelchair.
- Prevent the scooter from coming into contact with sand: sand may affect the moving parts of the wheelchair, resulting in unnecessarily rapid wear.
- Do not put the wheelchair in direct sunlight: the upholstery may become too hot, which may cause burning or an allergic reaction of the skin.
- Do not drive the wheelchair when you are under the influence of drugs, alcohol or medication that could affect your ability to drive.
- You must have sufficient eye sight to be able to safely drive the wheelchair.
- It is imperative to turn on the lights in case of reduced visibility.

- When driving at a high speed you must be extra careful. Select a lower maximum speed on sidewalks and in pedestrian areas.
- Be sure that no items of clothing hang loose. These could be caught between the wheels.
- When driving on a slope always make sure that all adjustment options are in the lowest position and that the legrests are in the lowest position.

### 3.6 Curves

- Never take curves at full speed. Decrease speed before taking a curve.
- Use the indicators when changing direction.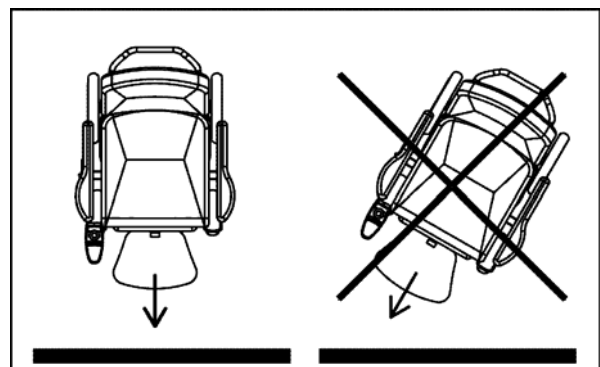

### 3.7 Obstacles

#### Driving up obstacles

1. Drive straight up to the kerb.
2. Slowly move the joystick forward. Drive up the sidewalk without changing direction.
3. As soon as the front wheels are on the sidewalk, you must keep your speed for the rear wheels to be able to move the sidewalk as well. If you cannot drive up onto the sidewalk, find a lower place where you can.

### Driving down obstacles

1. Drive straight up to the edge of the sidewalk.
2. Slowly move the joystick forward. Drive carefully down the sidewalk without changing direction.

**Do not drive down stairs!**

### 3.8 Slopes

#### Driving up slopes


- Only drive up a slope with all adjustment options in the lowest position and the legrests in the lowest position.
- Slopes with an angle of more than 14° (RWD) or 6° (FWD) must not be taken.
- Always drive slowly and with awareness on slopes.
- Do not drive down slopes with loose gravel or a sandy surface, as one of the rear wheels might slip
- Avoid sudden and jerky movements.
- Avoid directional changes on slopes.
- Do not turn on slopes.
- If you notice a strong decrease in speed when driving up a slope, take a less steep route.
- Driving on a slope too long can cause overheating.

#### Driving down slopes

- If you must reverse down a slope, make sure the speed control is in the lowest position and drive as slowly as possible and with the utmost control.
- It can be very dangerous to reverse down a slope.
- Prevent the wheelchair from gaining too much speed.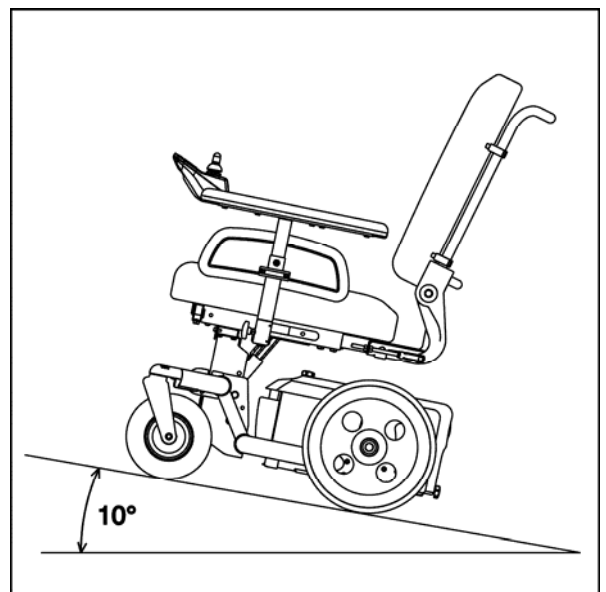

### 3.9 Reducing the wheelchair

#### Removing the seat

1. Pull up the seat (A) in the front.
2. When replacing the seat, the clips (B) must click around the frame (C) of the chair.

**Check if the seat is clicked in place securely.**

#### Folding down the backrest

1. Flip the lock clip (A) off the adjusting bar.
2. Pull the lock clip from the hinged joint.
3. Fold down the backrest.

#### Removing the armrest

Loosen the cap and remove the armrest.




**The controller remains connected to the wheelchair with the cable.**

#### Removing the legrest

1. Push the catch (A) backward to unlock the legrest.
2. Swing the legrest (B) outward.
3. Raise the legrest and remove it from the guide.

**Make sure you hear a click when you put the legrest back in place. The legrest is now securely locked.**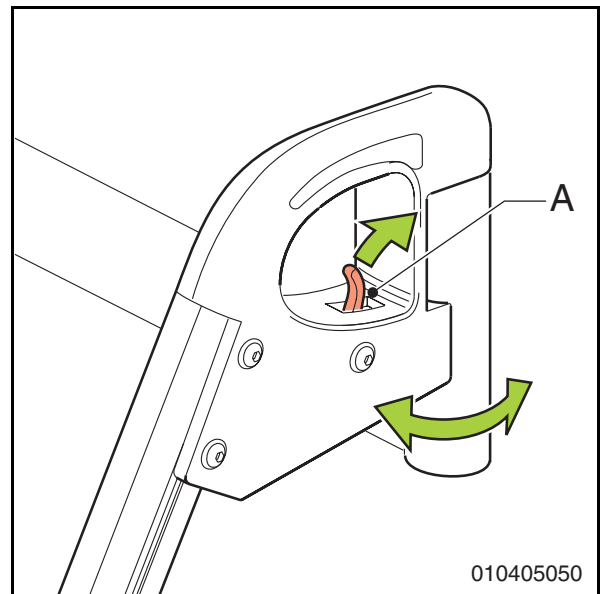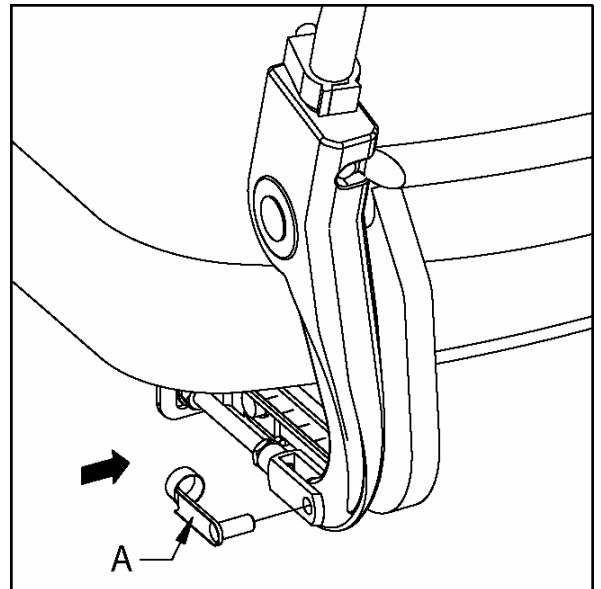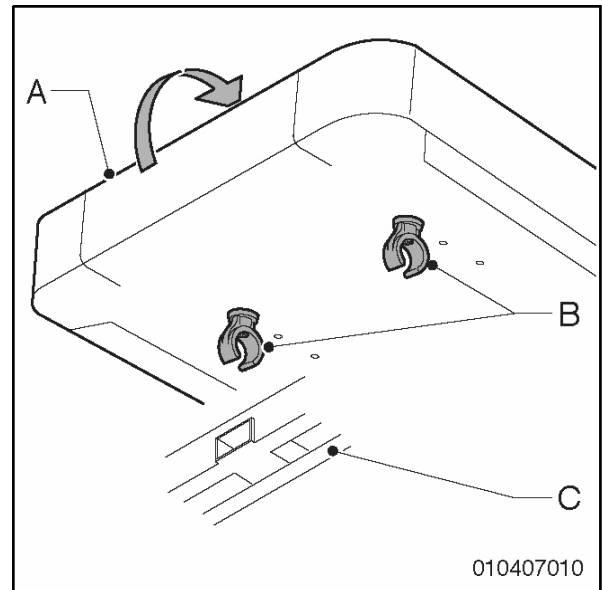

### 3.10 Transport

The wheelchair can be placed in a car. The best way to do this is by reducing the wheelchair or using steel ramps.


Handicare recommends wheelchair users to travel in a regular car seat whenever possible. A wheelchair is not designed as a car seat and cannot offer the level of safety that is offered by standard car seats, no matter how well the wheelchair is fixed to the vehicle concerned. As it is not possible for every wheelchair user to make a transfer, the wheelchair should be secured as described below.

- **The Ibis X-Series has been crash tested and has met all the pass requirements of ISO 7176-19.**
- **Your safety during transport largely depends on the person securing the wheelchair. It is your responsibility to ensure the wheelchair is secured as safely as possible.**
- **Always ask for confirmation of the transporter that the vehicle is suitably designed, insured and equipped to transport a person in a wheelchair.**
- **Parts of the wheelchair that can be easily detached should be removed and stored elsewhere during transportation.**
- **The wheelchair should always be transported in a forward facing direction.**
- **Handicare recommends an Unwin tie-down system that is tested and approved according to ISO10542.**
- **The wheelchair should be secured only at the tie-down points on the frame, not onto**

**any accessories (armrest, anti-tip brackets, etc.).**

- **Handicare recommends that the wheelchair be supplied with a headrest and that this should be used during transportation.**
- **During transportation it is essential that you are secured by a three point lap belt, which is anchored to the vehicle wall and floor rails. A lap belt in your wheelchair must not be considered as an alternative to the occupant restraint system.**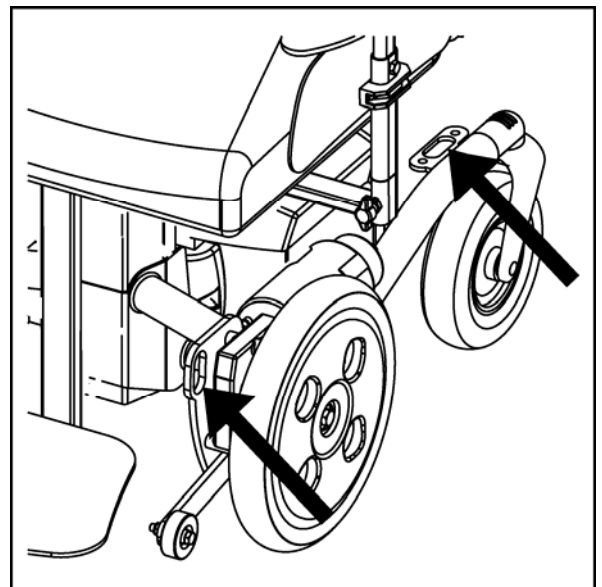


### 3.11 Adjustment options (dealer)

#### Seat height

Before adjusting the seat height, remove as many heavy seat parts as possible (seat, backrest, etc.) to limit the weight to be lifted.

1. Loosen the 4 bolts (A) using a 13 mm spanner, and remove them.
2. Lift the seat post off the bracket (B) and place it back in the desired position on the bracket.
3. Tighten the bolts (A) securely.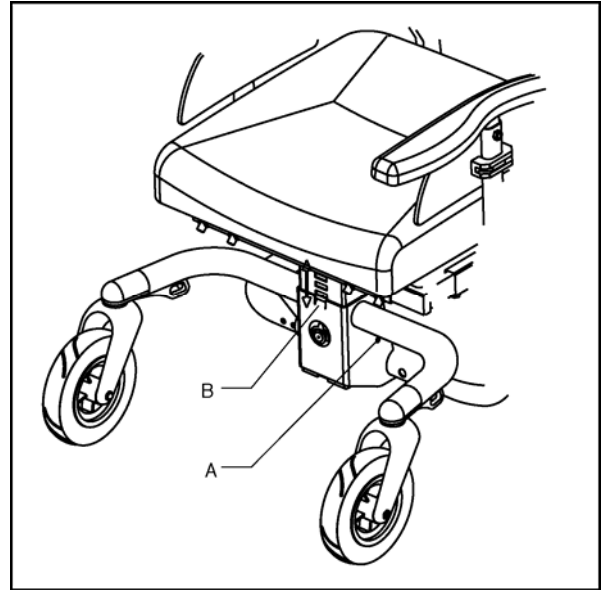


#### Seat angle

1. Loosen the bolt using a 13 mm spanner, and remove it.
2. Move the seat in the desired position and put the bolt back in place.

- A = 0°
- B = 2°
- C = 4°
- D = 6°
- E = 8°

3. Tighten the bolt securely.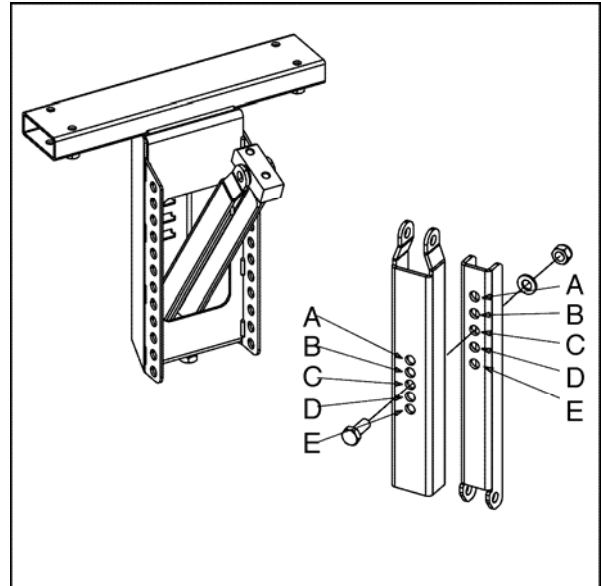


#### Controller 'fixed'

##### Depth:

1. Loosen the setscrew (B) a few turns with a 4mm Allen key.
2. Slide the controller (C) in the desired position.
3. Tighten the setscrew securely.

##### Height:

1. Loosen the setscrew (A) a few turns with a 3 mm Allen key.
2. Slide the controller (C) in the desired position.
3. Tighten the setscrew securely.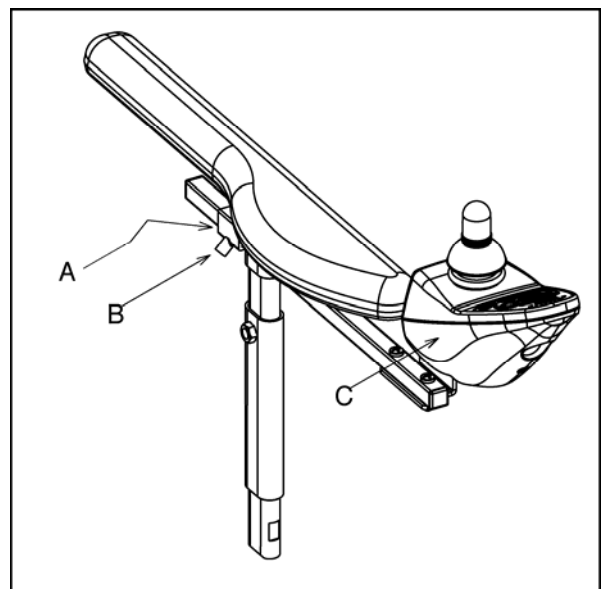


## Controller 'swing-away'

### Height

1. Loosen the setscrew (A) a few turns with a 3 mm Allen key.
2. Slide the controller with the swing-away mechanism in the desired position.
3. Tighten the setscrew securely.

### Depth

1. Loosen the setscrew (B) a few turns with a 4mm Allen key.
2. Slide the controller in the desired position.
3. Tighten the setscrew securely.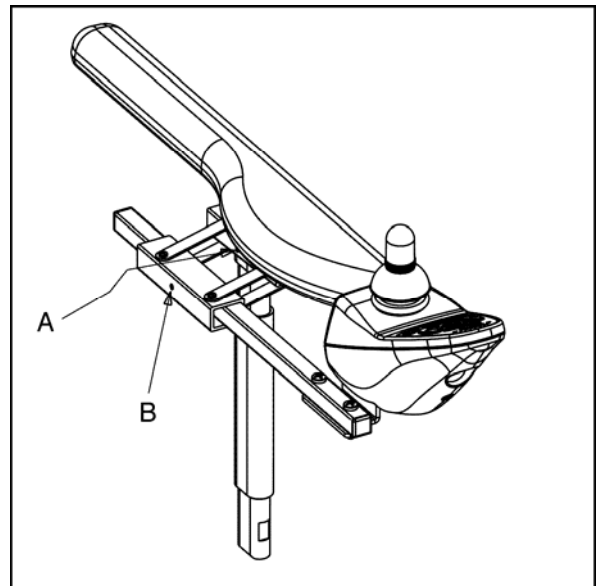


## Anti-tip brackets (option)

For extra safety anti-tip brackets are available for the Ibis XP models. These can be put in two different positions.


The anti-tip brackets will limit the obstacle height that can be taken with the wheelchair, see figure.

With low anti-tip brackets the wheelchair will tilt less in case of instability.

1. Loosen the bolts (B).
2. Put the anti-tip brackets in the desired position.
3. Tighten the bolts (B) securely.

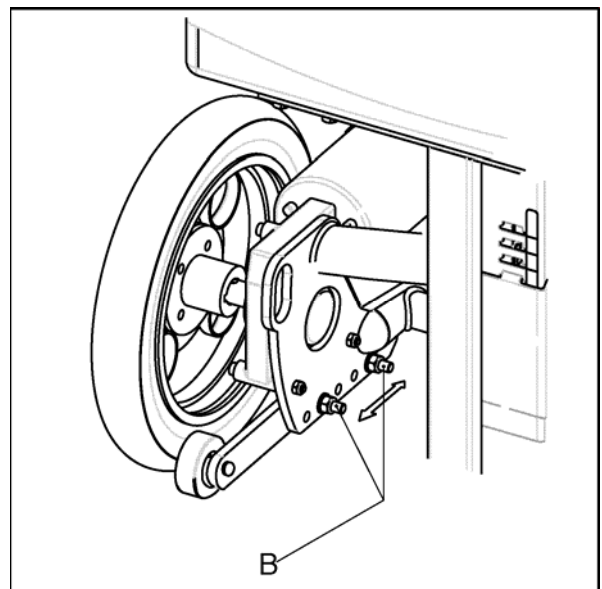**After adjusting the anti-tip brackets always check if they work properly by tilting the wheelchair forward or backward in a safe manner until the anti-tip brackets touch the ground.**
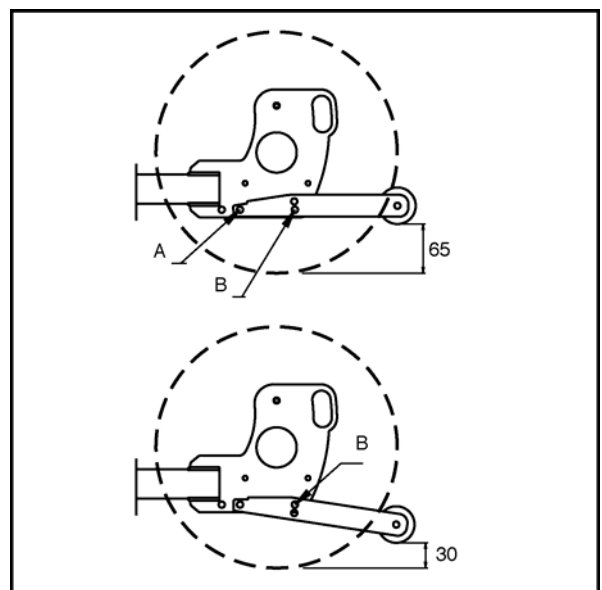
### 3.12 Electrical adjustment options


The wheelchair can be equipped with a maximum of 2 electrical adjustment options:

- Electrical backrest recline
- Electrical central legrest
- Electrical high/low
- Electrical seat tilt

If you want to choose between various adjustment options, you have to push the adjustment key (J) several times.

#### Adjusting:

1. Switch on the wheelchair.
2. Push the adjustment key (J). The indicator light of the key will go on. You can no longer drive the wheelchair.
3. Move the joystick forward: the adjustment option moves forward, or move the joystick backward: the adjustment option moves backward.
4. Push the adjustment key (J) to switch off the adjustment function.

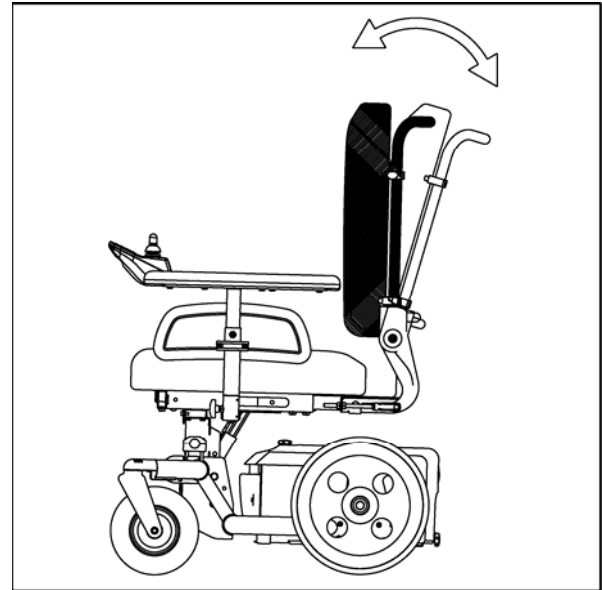## 4 Maintenance

### 4.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Daily	Charge the batteries after each use, see 4.2.
Weekly	Check tyre pressure and inflate the tyres if necessary, see 4.3.
Monthly	Clean the wheelchair, see 4.4. Check the batteries (in summer: once every fortnight; in winter: once a month), see 4.2.

### 4.2 Batteries

The wheelchair has 'dry' gel batteries. These dry batteries are entirely closed and maintenance free.


**The use of 'wet' batteries is not permitted.**

#### Charging the batteries

**Consult the owner's manual of the battery charger.**

In case of normal use the batteries should be charged every night.

1. Switch off the wheelchair.
2. Put the plug of the charge cord in the charge connection (A).
3. Put the plug of the battery charger in the wall outlet of the electricity mains.

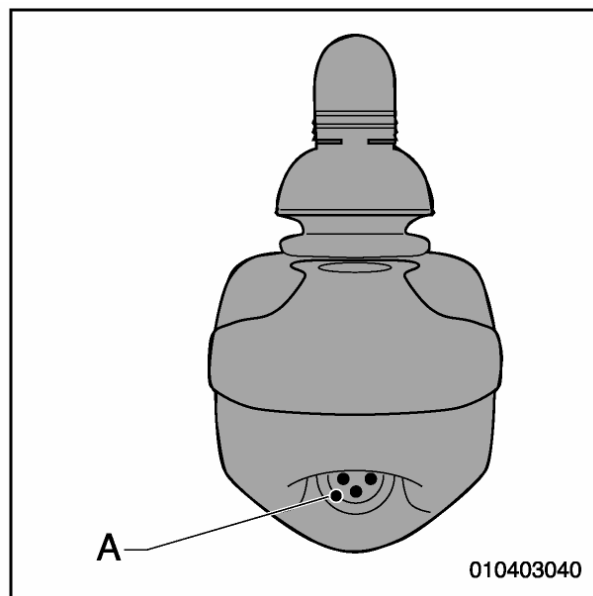When the batteries have been charged:

1. Remove the plug from the charge connection.
2. Remove the plug of the battery charger from the wall outlet of the electricity mains.

- **As soon as the plug of the charge cord is connected to the charge connection, the operation of the wheelchair is switched off.**
- **Always remove the charge cord when the batteries have been charged. This way, you will prevent slow depletion of the batteries.**

### Maintenance of the batteries

Refer to the battery instructions.

- **Make sure that the batteries are always well charged.**
- **Do not use the wheelchair if the batteries are almost depleted and never completely deplete the batteries. This can seriously damage the batteries and you run the risk to come to an unintended standstill.**

### Cleaning the batteries

Keep the batteries clean and dry. Dirt and water can cause a leak, reducing the capacity of the batteries. After cleaning grease the poles with acid-free Vaseline.

### Replacing the batteries

If the capacity of the batteries is reduced to the extent that the wheelchair can only make very short trips, the batteries are at the end of their lifespan. The batteries must be replaced as soon as possible. Contact your dealer for this.

### 4.3 Checking and inflating the tyres

- **Make sure the tyres are kept at the correct pressure.**
- **The tyre pressure may never exceed the maximum value indicated on the tyre.**

Inflate the tyres with a foot pump or a bicycle pump, in combination with the reducing nipple included in the delivery. You can also have your tyres inflated at your local petrol station.

- Tyre pressure drive wheel: 2.7 bar
- Tyre pressure castor wheel: 2.5 bar

### 4.4 Cleaning the wheelchair

#### Removing dry dirt

Upholstery, metal parts and frame parts can usually easily be cleaned with a dry soft cloth.

#### Removing mud and other wet dirt

The dirty parts can best be cleaned by first wiping them with a wet sponge and then wiping them dry with a dry soft cloth.


- **Never use abrasive or aggressive cleansers. They can scratch the wheelchair.**
- **Do not use organic solvents such as thinner, benzene or white spirit.**
- **Be careful with water in view of the electronic system.**

## 5 Troubleshooting

### 5.1 Troubleshooting table

If the wheelchair will not function, while the batteries are fully charged, check the following points before contacting your dealer.

- Switch off the wheelchair and then switch it on again. Check if the problem has been solved.
- Check if all charge plugs are properly connected and secured.
- Check if the wheelchair is in the DRIVE position (not in NEUTRAL).
- Check if the position of the speed control has not changed.

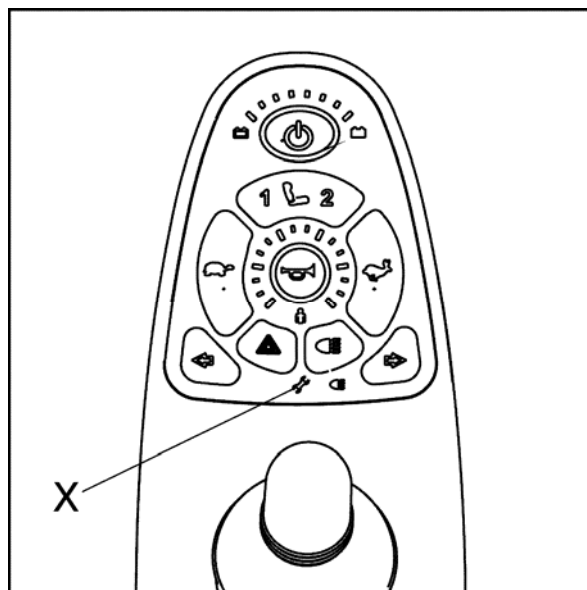In case of a malfunction the battery indicator lights will be flashing on the (Shark controller). On the Shark lights and Shark seats-lights controllers the spanner symbol will be flashing (X). Use the table below to find out the cause and report this information to your dealer.

Flashing frequency indicator lights	Fault description	Action
5	Motor 1 is in the NEUTRAL position	Put the free wheel lever of motor 1 in the DRIVE position
6	Motor 2 is in the NEUTRAL position	Put the free wheel lever of motor 2 in the DRIVE position

In all other cases contact your dealer.

## 6 Technical specifications

Description	
Model	Ibis XP
Max. user weight	160 kg
Total length	1040 mm
Total width	595 mm
Total weight excl. batteries	60 kg
Static stability downward	15°
Static stability upwards	15°
Static stability sideways	15°
Range (ISO 7176) 33 Ah/c5 batteries	25 km
Range (ISO 7176) 40 Ah/c5 batteries	31 km
Dynamic stability on slopes FWD	6°
Dynamic stability on slopes RWD	14°
Climbing capacity for obstacles FWD	70 mm
Climbing capacity for obstacles RWD	50 mm
Climbing capacity for obstacles RWD with kerb climber	100 mm
Maximum speed	6 km/h
Seat tilt	15°
Seat angle	0 - 6°
Effective seat depth	440 – 520 mm
Effective seat width	380 – 555 mm
Seat height front	435 – 540 mm
Backrest angle	89 - 120°
Backrest height	520 – 570mm
Lower leg length	390 – 500 mm
Armrest height	180 – 280 mm
Front armrest to backrest	370 – 450 mm
Minimum turning radius RWD	790 mm
Minimum turning radius FWD	600 mm
Ground clearance	60 mm

Wheels	
Diameter castor wheel	8"
Diameter drive wheel	12,5"
Tyre pressure castor wheel	2,5 Bar
Tyre pressure drive wheel	2,7 Bar

Batteries	
Maximum dimensions batteries FWD (WxDxH)	278x200x180
Maximum dimensions batteries RWD (WxDxH)	278x358x180
Maximum battery capacity	30 / 40 Ah
Maximum permissible charging voltage	24 V
Maximum charging current	10 A
Connector type	A DIN 72311

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Handicare B.V.