



Yes, you can.®

➤ Invacare® **Spectra**® XTR

Freedom of movement

The *Invacare*® **Spectra**® XTR has been developed to combine powerful driving performance with comfort. High torque motors and a unique suspension design provide a smoother, easier ride. The interchangeable seating systems offer unrivalled flexibility and modularity, this feature coupled with a new easy to remove cable free battery system makes the **Spectra**® XTR extremely quick and simple to service.

NEW





➤ **Driving performance and comfort**

The **Spectra[®]XTR** has a specially designed suspension system. The springs are mounted centrally under the seat and attach to a long swing arm which improves traction and enhances outdoor performance and comfort. **Spectra[®]XTR** is available in two speeds 6 and 10 km/h.

➤ **Compactness and manoeuvrability**


Excellent indoor mobility is obtained due to the particularly compact dimensions of the **Spectra[®]XTR**. When choosing smaller castors, this feature is optimized, providing a width of below 59 cm.

➤ **Modularity – functional and adaptable**

Powered wheelchairs are often used for long periods of time, with three interchangeable seating systems offering a wide range of seating options the **Spectra[®]XTR** offers a tailor made solution for comfort and positioning.

➤ **Serviceability and transportability**

With a clever and robust mechanism for detaching the seat from the wheelbase, transportation, storage and servicing are rendered refreshingly trouble-free. Just one cable has to be disconnected when taking the **Spectra[®]XTR** apart. To make servicing even easier, the cable free batteries slide out smoothly and without effort.

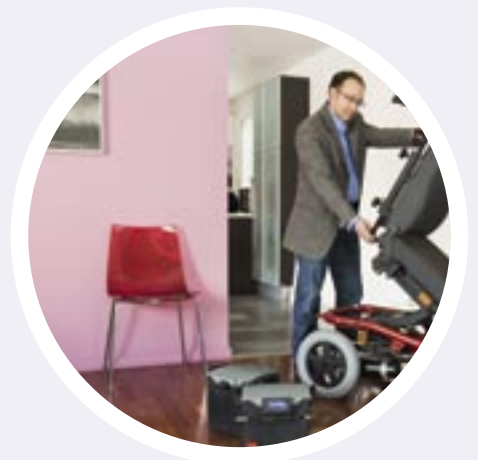

Features & Options



Removable seat unit

Pull a strap, lift the seat and disconnect the remote cable in order to take the seat off. It couldn't be easier!



Lock back kerb rider




Removable batteries

Separate cable free battery boxes slide in and out without any effort. High-end sockets connect automatically.




Different applications

Choice of two castor sizes depending on customer requirements.

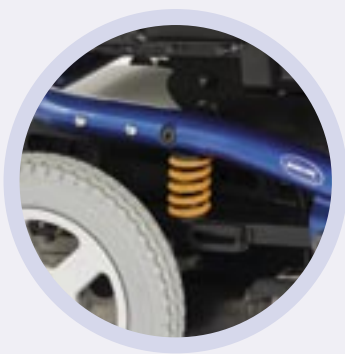

Spring for suspension

A comfortable drive and safe traction are ensured even at high speeds.

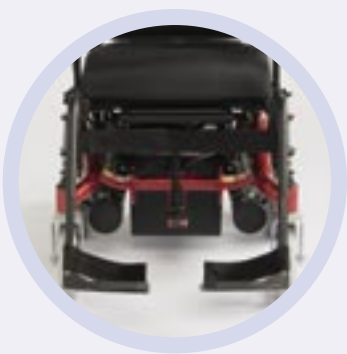

Narrow width

The total width with the option of small castors is below 59 cm.



Swing out anti tippers

A useful feature for storing the wheelbase.

Options



Standard seat

Features tension adjustable backrest for individual support.



Firm seat unit

Padded robust seating. Cross compatible with the well known **Spectra**® seat.



Flex2 seat

A comfort style seat supporting good posture. Cross-compatible to well known Rea® seat.


Technical data

Standard Seat	44 - 48 cm ¹	41, 46, 51 cm	46 cm ⁴	48 / 54 cm ⁴	29 - 36 cm ⁴	29 - 47 cm ^{4,5}
Flex2 Seat	39 - 43, 44 - 48, 49 - 53 cm ²	43 - 51 cm	46 cm ⁴	56 cm ⁴	24 - 31 cm, 26 - 34 ⁴	29 - 47 cm ^{4,5}
Firm Seat	44 - 48, 40 - 44, 34 - 37 cm ³	41, 46, 51 cm	46 cm ⁴	43 / 57 cm ⁴	19 - 32 cm ⁴	20 - 48 cm ^{4,5}

Standard Seat	-2°, 10°, 22°, 34°, 46°	59 / 62 cm ⁶	94 cm	116 cm ⁷	82 / 84 ⁶	98 kg ¹⁰
Flex2 Seat	0° - 30°	59 / 62 cm ⁶	98 cm	116 cm ⁷	82 / 84 ⁶	94 kg ¹⁰
Firm Seat	0° - 30°	59 / 62 cm ⁶	105 cm	116 cm ⁷	82 / 84 ⁶	91 kg ¹⁰



50 Ah

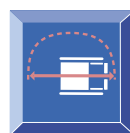

160 cm




6 cm⁸



15,8 %



2 x 230 W

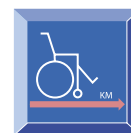

30 km⁹



6 / 10 km/h

- 1 seat cushion 43 cm
- 2 seat cushions 38 cm, 43 cm, 48 cm
- 3 seat upholstery 43 cm, 39 cm, 33 cm
- 4 measured without seat cushion / upholstery
- 5 certain legrest lengths are restricted to a special legrest type
- 6 depending on castor size
- 7 incl. standard legrests
- 8 10 cm with kerb climber
- 9 according to ISO 7176
- 10 total weight is depending on configuration

Frame colours



Black



Transparent blue



Electric red



Silver metallic

All measurement have been done under the Invacare® standard test conditions, unless quoted differently. All information quoted is believed to be correct at time of print. Invacare® reserves the right to alter product specifications without prior consultation.



INVACARE® Limited
 South Road · Bridgend Industrial Estate
 BRIDGEND CF31 3PY
 Tel. +44 (0) 1656 647327 · Fax +44 (0) 1656 649016
 E-mail: uk@invacare.com · www.invacare.co.uk

INVACARE® Ireland Ltd
 Unit 5 · Seatown Business Campus
 Seatown Road · SWORDS · County Dublin
 Tel. (353) 181 070 84 · Fax (353) 181 070 85
 Email: eire@invacare.com · www.invacare.ie